


DRILL LOG 1.0
ICX-3.3 IMPLANT
PREMIUM & ACTIVE MASTER

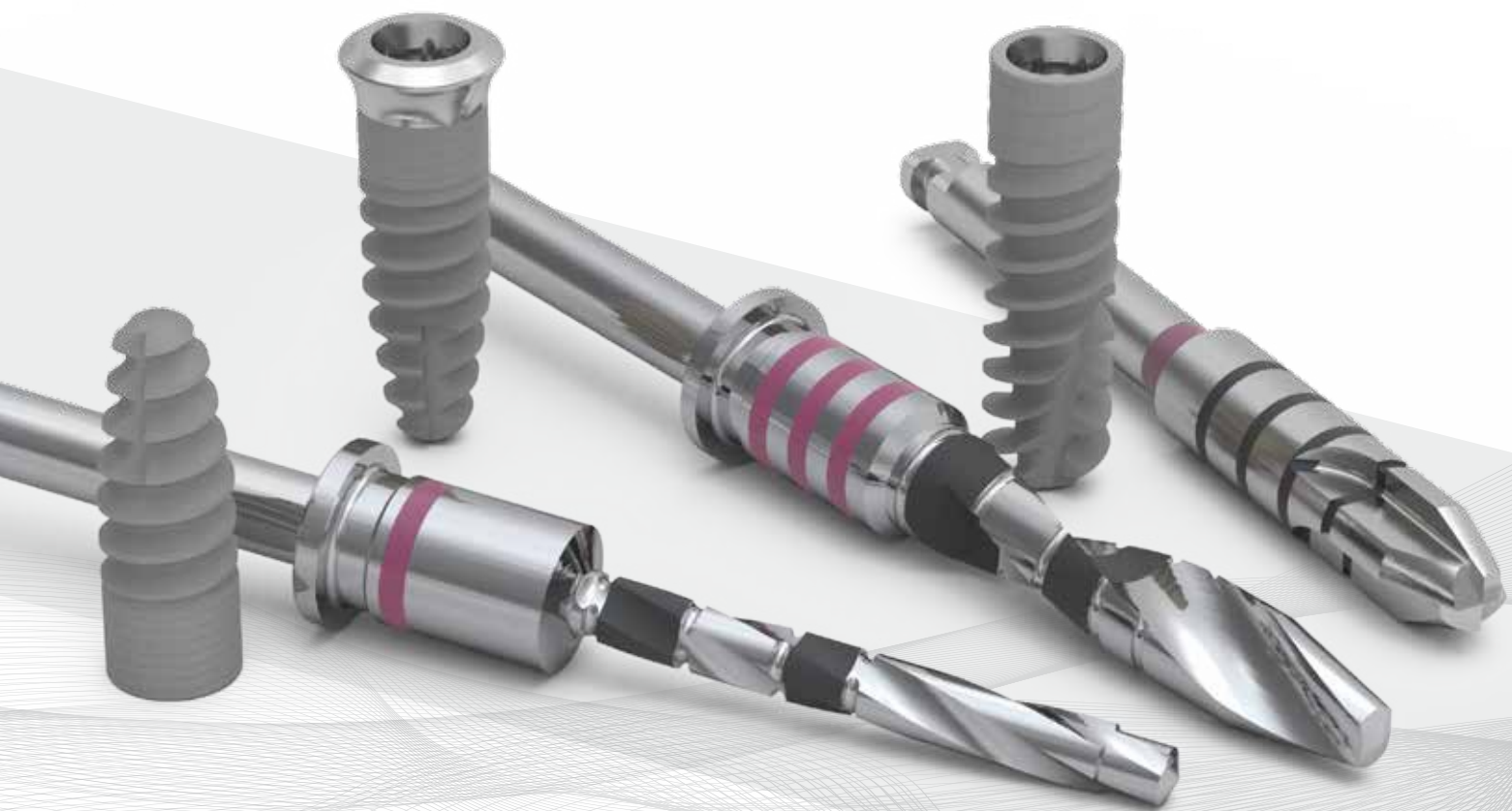

BE SMART. BE ICX.

medentis[®]
medical

**... THE POWER,
TO DECLARE VISIONS TO BE REALITIES.**



ICX-SHOP!



BE SMART. BE ICX.



Content

Page

ICX-3.3 Bonelevel (BL) & Tissuelevel (TL)-Implants & Surgery-Boxes 4-7

ICX-PREMIUM 3.3 BL & TL IMPLANTS - DRILL LOG 8

Surgical procedure **ICX-PREMIUM**

BL & TL IMPLANTS Ø 3,3mm.....9-12

ICX-ACTIVE MASTER 3.3 BL & TL IMPLANTS - DRILL LOG14

Surgical procedure **ICX-ACTIVE MASTER**

BL & TL IMPLANTS Ø 3,3mm.....15-18

ICX-3.3 IMPLANTS

BL & TL PREMIUM & ACTIVE MASTER


DIGITAL. DENTAL. FINAL.

medentis
medical


ICX-3.3 IMPLANT

BL & TL PREMIUM & ACTIVE MASTER

In areas with limited space, narrow dental implants offer clear advantages. This is due to the development of innovative implant types, with a reduced diameter, which make it possible to meet minimum criteria or a professional indication.

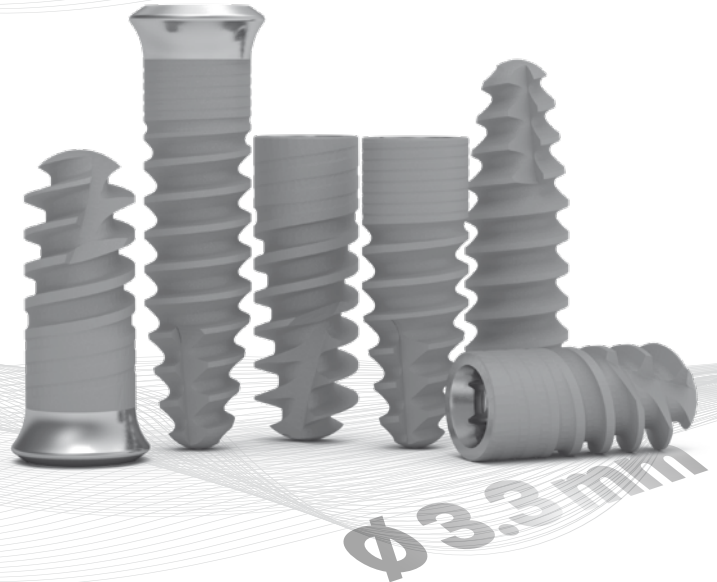


OUTSTANDING PROPERTIES OF THE ICX-Ø 3.3mm IMPLANTS

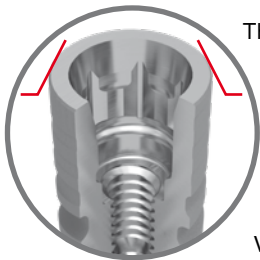


The ICX-PREMIUM- Ø 3.3mm and the ICX-ACTIVE-MASTER Ø 3.3mm implant open up completely new horizons in the field of reduced-diameter implantation!

We are expanding our ICX family with a reduced-diameter implant. This means that additional indications can be covered with the usual high quality and precision of the ICX-typical implant-abutment connection.

For this reason, we have reconsidered this for the rethought and developed a completely new prosthetic line on the prosthetic line.

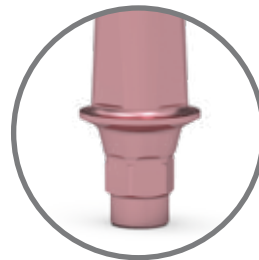

Our aim was to provide you as a practitioner with an implant, which can be used in borderline cases without having to compromise on stability.

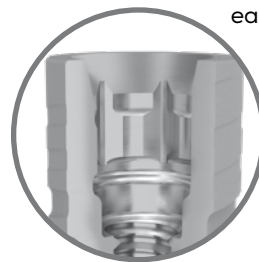

So you can now solve difficult cases more easily and rely on our ICX-PREMIUM Ø 3.3mm and the ICX-ACTIVE-MASTER Ø 3.3mm implants!

IMPORTANT NOTICE:

The ICX-PREMIUM Ø 3.3mm and the ICX-ACTIVE-MASTER Ø 3.3mm implants can **NOT** be restored with the ICX standard prosthetics!

The ICX-Ø 3.3mm implants have a different internal geometry of the ICX-PREMIUM & ICX-ACTIVE-MASTER ICX-ACTIVE-MASTER red, Ø 3,75mm - green, Ø 4,1mm - blue, Ø 4.8mm implants have their own prosthetic line. (system colour pink)!



ICX-BL & TL Ø 3,3mm Implants

ICX-PREMIUM Ø 3.3mm Implants (BL)

All ICX-PREMIUM Ø 3.3mm implants are only available in a pack of 6

Article-No.

C-001-330080	ICX-PREMIUM Ø 3.3 · L 8mm (6 pcs.)
C-001-330100	ICX-PREMIUM Ø 3.3 · L 10mm (6 pcs.)
C-001-330125	ICX-PREMIUM Ø 3.3 · L 12.5mm (6 pcs.)

Ø 3.3mm




ICX-ACTIVE-MASTER Ø 3.3mm Implants (BL)

All ICX-ACTIVE-MASTER Ø 3,3mm implants are only available in a pack of 6

Article-No.

ACM-001-330080	ICX-ACTIVE-MASTER Ø 3.3 · L 8mm (6 pcs.)
ACM-001-330100	ICX-ACTIVE-MASTER Ø 3.3 · L 10mm (6 pcs.)
ACM-001-330125	ICX-ACTIVE-MASTER Ø 3.3 · L 12.5mm (6 pcs.)

Ø 3.3mm

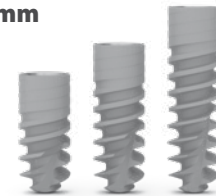


ICX-PREMIUM Ø 3.3mm Implants (TL)

All ICX-PREMIUM Ø 3,3mm implants are only available in a pack of 6

Article-No.

T-001-330080	ICX-PREMIUM TissueLine Implantat Ø 3.3 x 8mm (6 pcs.)
T-001-330100	ICX-PREMIUM TissueLine Implantat Ø 3.3 x 10mm (6 pcs.)
T-001-330125	ICX-PREMIUM TissueLine Implantat Ø 3.3 x 12.5mm (6 pcs.)

Ø 3.3mm

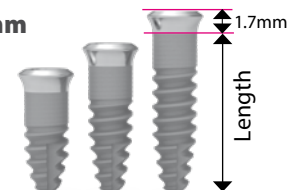


ICX-ACTIVE-MASTER Ø 3.3mm Implants (TL)

All ICX-ACTIVE-MASTER Ø 3,3mm implants are only available in a pack of 6


Article-No.

TAM-001-330080	ICX-ACTIVE MASTER TissueLine Implantat Ø 3.3 x 8mm (6 pcs.)
TAM-001-330100	ICX-ACTIVE MASTER TissueLine Implantat Ø 3.3 x 10mm (6 pcs.)
TAM-001-330125	ICX-ACTIVE MASTER TissueLine Implantat Ø 3.3 x 12.5mm (6 pcs.)


Ø 3.3mm

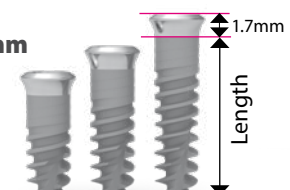


ICX-PREMIUM Ø 3.3mm Drills

Article-No.	ICX-PREMIUM Ø 3.3 drills	Article
C-014-003330		ICX-PREMIUM Parallel drill soft Ø 3.3 pink I ●
C-014-006330		ICX-PREMIUM Parallel drill standard Ø 3.3 pink II ●
C-014-007330		ICX-PREMIUM Parallel drill hard Ø 3.3 pink III ●
C-014-005330		ICX-PREMIUM STOP drill Ø 3.3 pink ●


ICX-PREMIUM Surgery-Box

Article-No.	ICX-PREMIUM Surgery Box (filled)	
C-018-100003	ICX-PREMIUM Surgery Box (filled)	


ICX-ACTIVE-MASTER Ø 3.3mm Drills

Article-No.	ICX-ACTIVE-MASTER Ø 3.3 drills	Article
ACM-014-003330		ICX-ACTIVE-MASTER Parallel drill soft Ø 3.3 pink I ●
ACM-014-006330		ICX-ACTIVE-MASTER Parallel drill standard Ø 3.3 pink II ●
ACM-014-007330		ICX-ACTIVE-MASTER Parallel drill hard Ø 3.3 pink III ●
ACM-014-005330		ICX-ACTIVE-MASTER STOP drill Ø 3.3 pink ●

ICX-ACTIVE-MASTER Surgery-Box


Article-No.	ICX-ACTIVE-MASTER Surgery Box (filled)	
ACM-018-000006	ICX-ACTIVE-MASTER Surgery Box (filled)	

ATTENTION! ++ IMPORTANT NOTICE ++ ATTENTION!

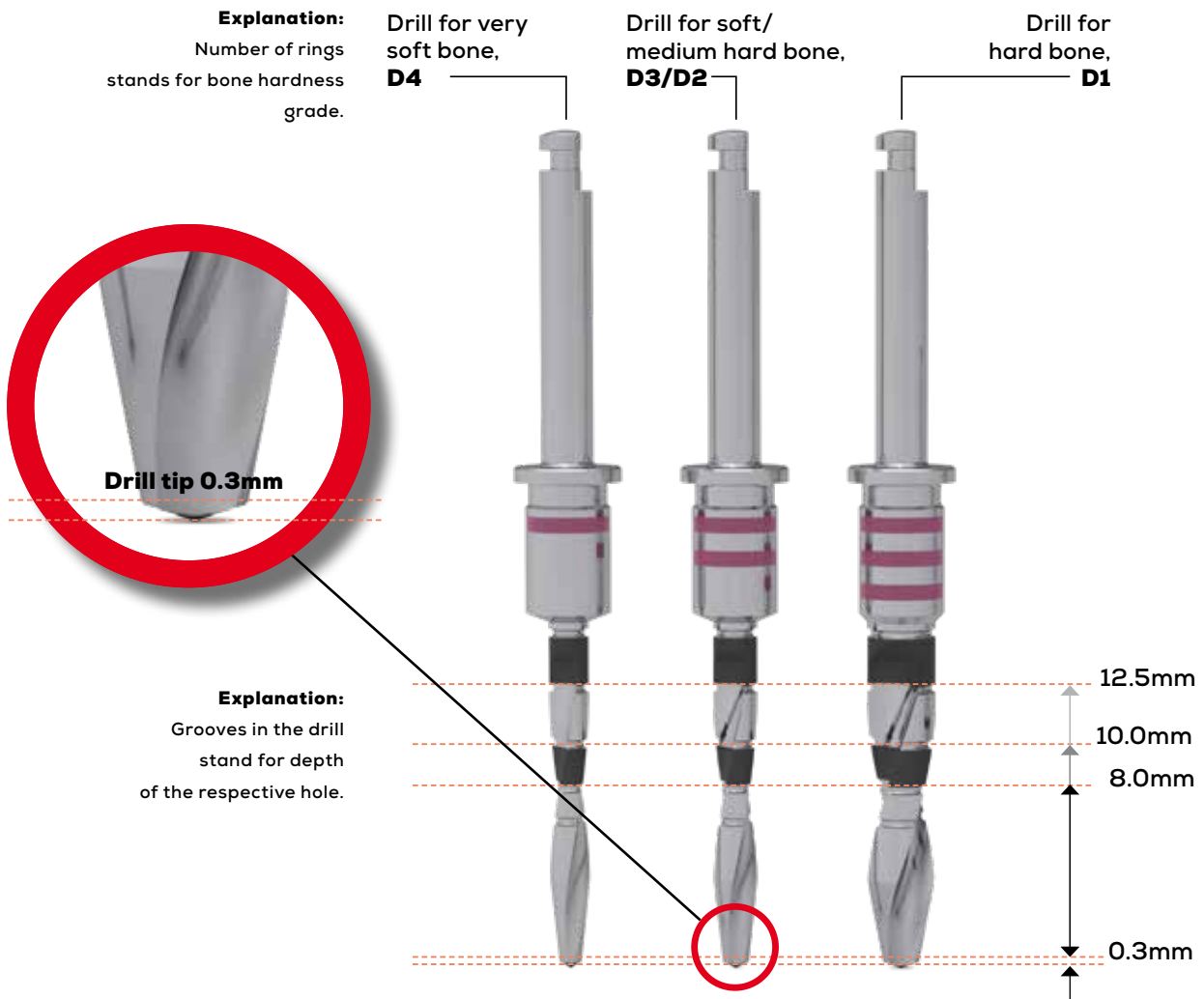

With the ICX-Ø3.3mm drills, the drill length is 0.3mm longer due to the drill tip 0.3mm longer - It must be ensured, that this additional length is permissible, when working in the area of important anatomical structures.

medentis
medical

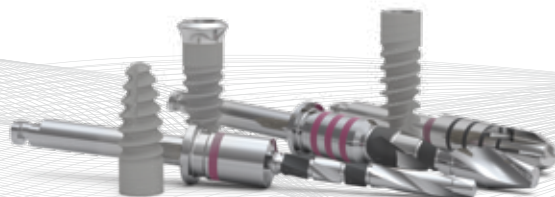

BRIEF OVERVIEW- DRILL LOG 1.0

for ICX-PREMIUM 3.3

BL & TL Ø 3.3mm Implants

SOFT BONE

Step	Description	Article number
1	Cortical Drill Pre-drill	FIL-186RF
2	ICX-PREMIUM 3.3 Parallel drill soft I	C-014-03330
3	ICX-PREMIUM 3.3 Stop drill	C-014-005330

MEDIUM HARD BONE

Step	Description	Article number
1	Cortical Drill Pre-drill	FIL-186RF
2	ICX-PREMIUM 3.3 Parallel drill standard II	C-014-006330
3	ICX-PREMIUM 3.3 Stop drill	C-014-005330

HARD BONE


Step	Description	Article number
1	Cortical Drill Pre-drill	FIL-186RF
2	ICX-PREMIUM 3.3 Parallel drill standard II	C-014-006330
3	ICX-PREMIUM 3.3 Parallel drill dard III	C-014-007330
4	ICX-PREMIUM 3.3 Stop drill	C-014-005330

ICX-3.3 IMPLANT PREMIUM

ICX-PREMIUM

BL & TL Ø 3.3mm Implants


DRILL LOG 1.0
for ICX-PREMIUM
BONELEVEL (BL) &
TISSUE LEVEL (TL)

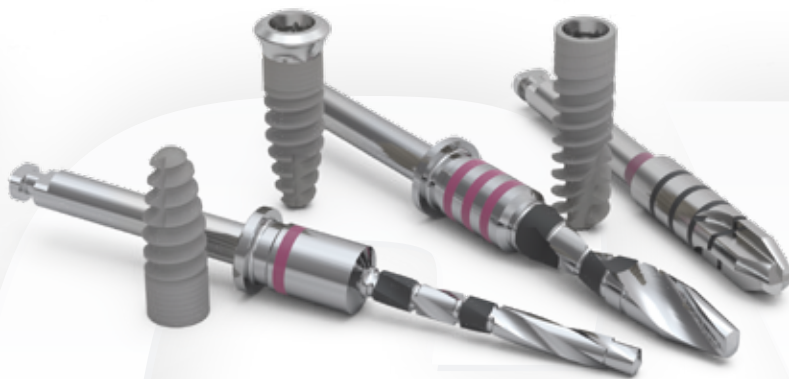

ICX-PREMIUM

ICX-PREMIUM BL & TL Ø 3.3mm Implants

Surgical procedure **ICX-PREMIUM** BL & TL Ø 3.3mm

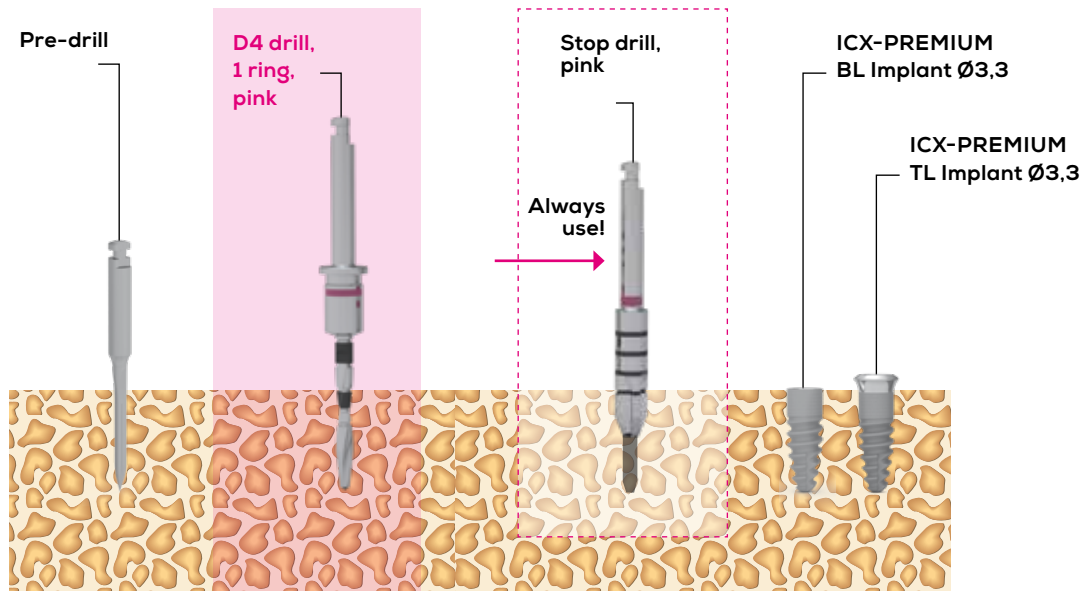

ICX-PREMIUM STANDARD IMPLANTATION

DRILL PROTOCOL FOR VERY SOFT BONE

PLEASE ONLY USE DRILLS WITH 1 COLOUR RING HERE!




**Important note: Drill for very soft bone D4,
recognise by the colour marking = 1 pink ring**



ICX-PREMIUM BL & TL Ø 3.3mm Implants

Surgical procedure **ICX-PREMIUM** BL & TL Ø 3.3mm

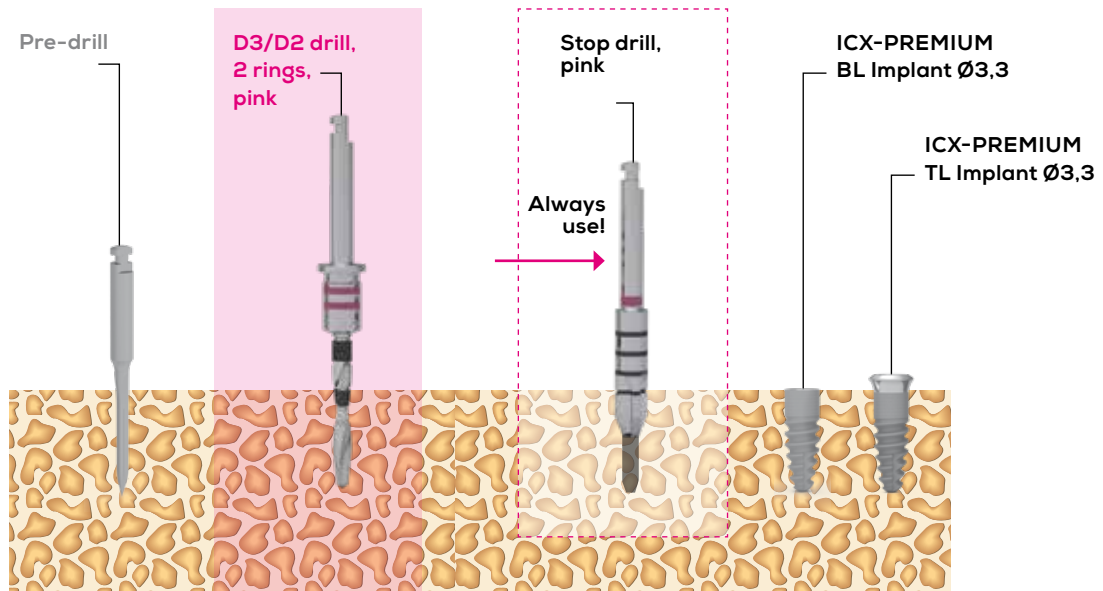

ICX-PREMIUM STANDARD IMPLANTATION

DRILL PROTOCOL FOR SOFT/MEDIUM HARD BONE
PLEASE ONLY USE DRILLS WITH 2 COLOUR RINGS HERE!




Important note: Drill for soft/medium hard bone, D3/D2 can be recognised by the colour marking = 2 pink rings



ICX-PREMIUM BL & TL Ø 3.3mm Implants

Surgical procedure **ICX-PREMIUM** BL & TL Ø 3.3mm

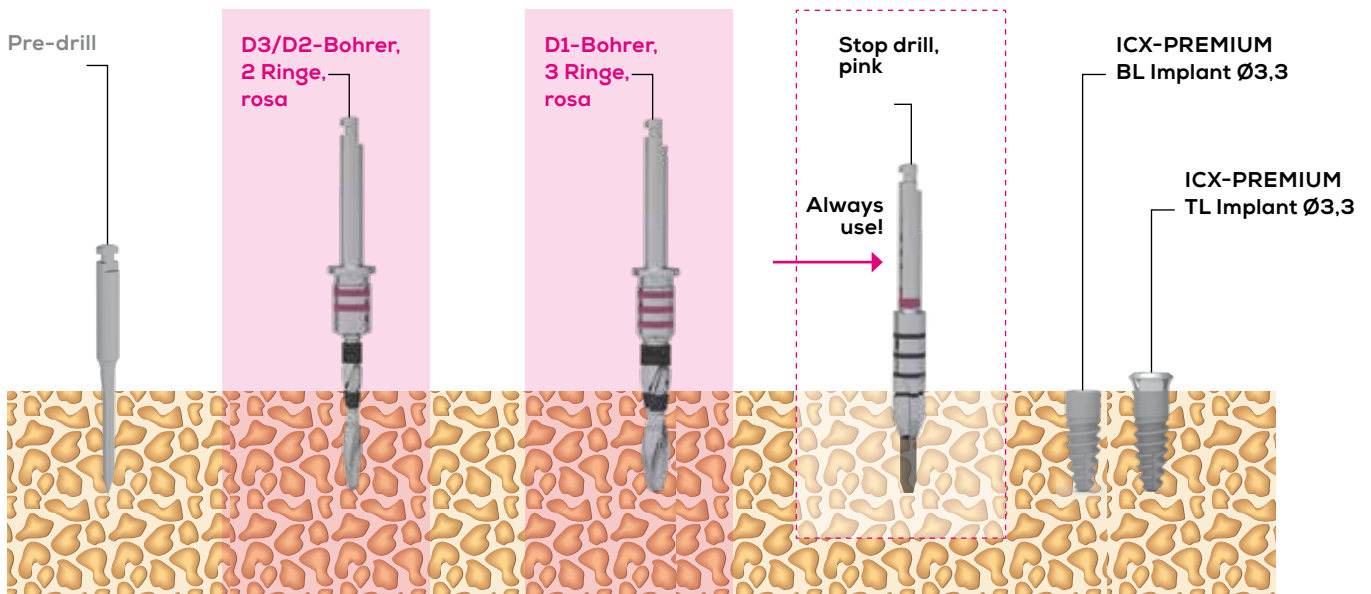

ICX-PREMIUM STANDARD IMPLANTATION

DRILL PROTOCOL FOR HARD BONE

PLEASE ONLY USE DRILLS WITH 3 COLOUR RINGS HERE!





**Important note: Drill for hard bone D1,
recognise by the colour marking = 3 pink rings**



ICX-3.3 IMPLANTS

BL & TL PREMIUM & ACTIVE MASTER

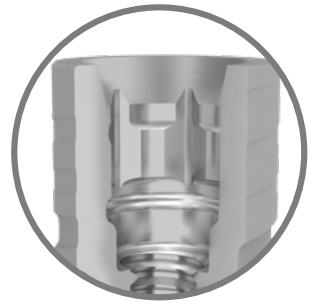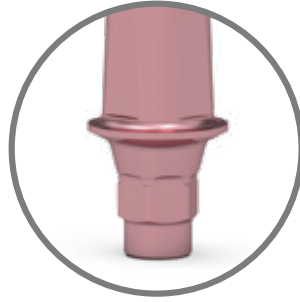

POWERFUL & PREDICTABLE!



**BE SMART.
BE ICX.**

BRIEF OVERVIEW- DRILL LOG 1.0

for ICX-ACTIVE MASTER

BL & TL Ø 3.3mm Implants

SOFT BONE

Step	Description	Article number
1	Cortical Drill Pre-drill	FIL-186RF
2	ICX-ACTIVE 3.3 Parallel drill soft I	ACM-014-003330
3	ICX-ACTIVE 3.3 Stop drill	ACM-014-005330

MEDIUM HARD BONE

Step	Description	Article number
1	Cortical Drill Pre-drill	FIL-186RF
2	ICX-ACTIVE 3.3 Parallel drill standard II	ACM-014-006330
3	ICX-ACTIVE 3.3 Stop drill	ACM-014-005330

HARD BONE

Step	Description	Article number
1	Cortical Drill Pre-drill	FIL-186RF
2	ICX-ACTIVE 3.3 Parallel drill standard II	ACM-014-006330
3	ICX-ACTIVE 3.3 Parallel drill standard hard III	ACM-014-007330
4	ICX-ACTIVE 3.3 Stop drill	ACM-014-005330


ICX-3.3 IMPLANT ACTIVE MASTER



ICX-ACTIVE MASTER

BL & TL Ø 3.3mm Implants


DRILL LOG 1.0
for ICX-ACTIVE MASTER
BONELEVEL (BL) &
TISSUE LEVEL (TL)

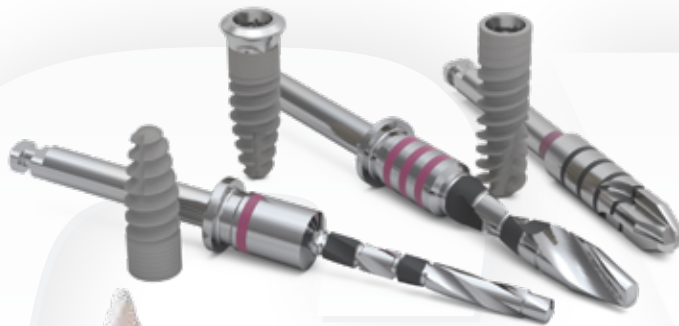

ICX-ACTIVE MASTER

ICX-ACTIVE MASTER BL & TL \varnothing 3.3mm Implants

Surgical procedure **ICX-ACTIVE MASTER** BL & TL \varnothing 3.3mm

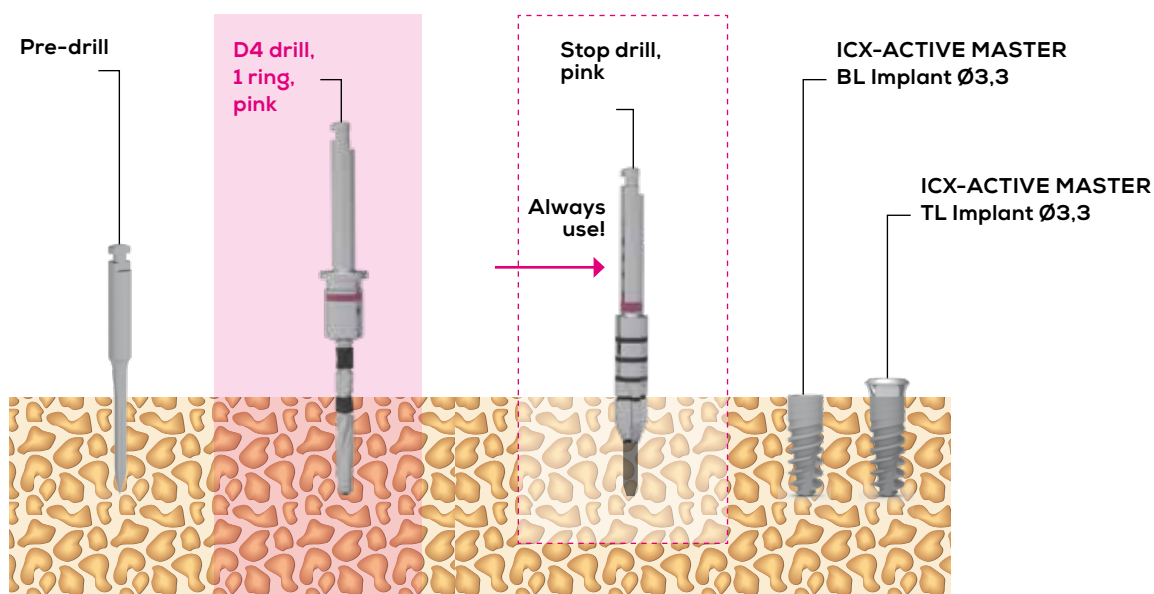

ICX-ACTIVE MASTER STANDARD IMPLANTATION

DRILL PROTOCOL FOR VERY SOFT BONE

PLEASE ONLY USE DRILLS WITH 1 COLOUR RING HERE!




Important note: Drill for very soft bone D4, recognise by the colour marking = 1 pink ring



ICX-ACTIVE MASTER BL & TL Ø 3.3mm Implants

Surgical procedure **ICX-ACTIVE MASTER** BL & TL Ø 3.3mm

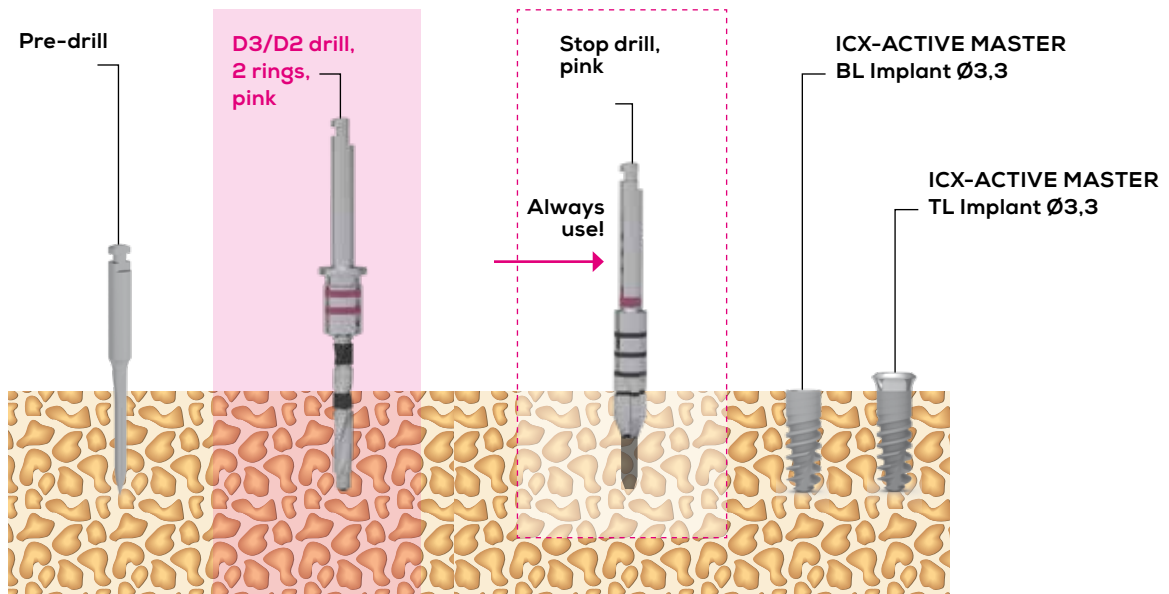

ICX-ACTIVE MASTER STANDARD IMPLANTATION

DRILL PROTOCOL FOR SOFT/MEDIUM HARD BONE
PLEASE ONLY USE DRILLS WITH 2 COLOUR RINGS HERE!




Important note: Drill for soft/medium hard bone, D3/D2 can be recognised by the colour marking = 2 pink rings



ICX-ACTIVE MASTER BL & TL Ø 3.3mm Implants

Surgical procedure **ICX-ACTIVE MASTER BL & TL Ø 3.3mm**

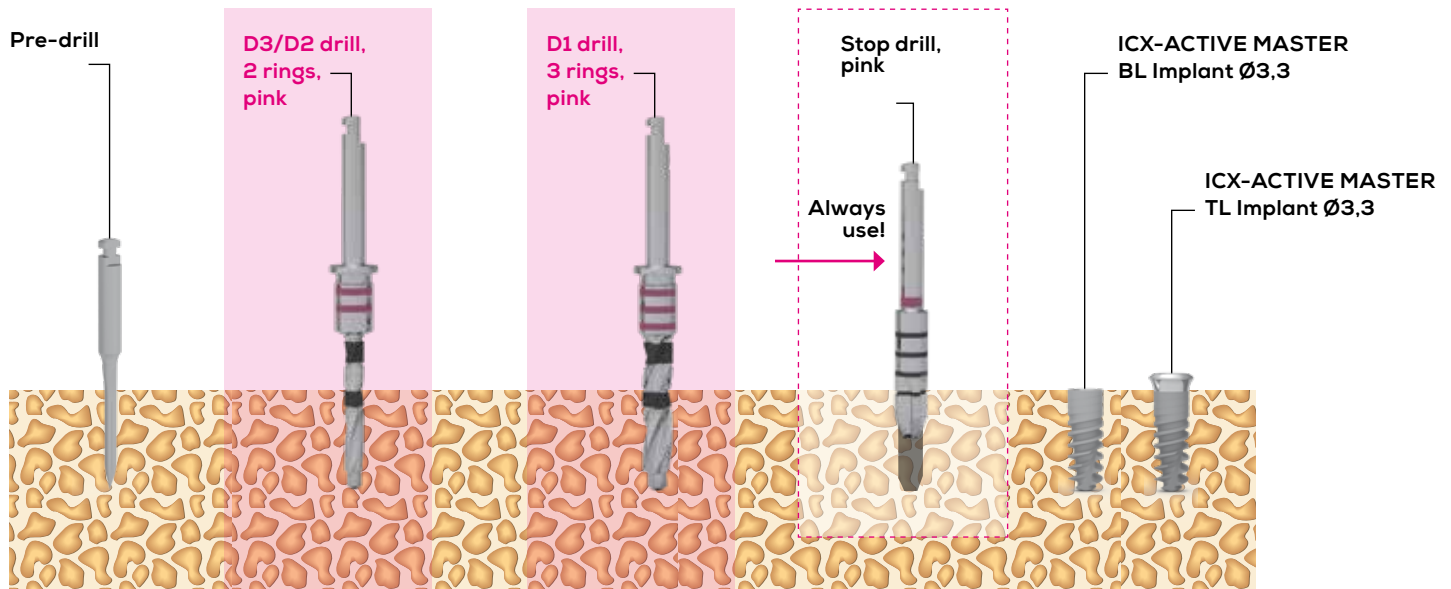

ICX-ACTIVE MASTER

STANDARD IMPLANTATION

DRILL PROTOCOL FOR HARD BONE

PLEASE ONLY USE DRILLS WITH 3 COLOUR RINGS HERE!



**Important note: Drill for hard bone D1,
recognise by the colour marking = 3 pink rings**



medentis[®]
medical

***... THE POWER,
TO DECLARE VISIONS TO BE REALITIES.***



DRILL LOG 1.0

ICX-3.3 IMPLANT

BL & TL PREMIUM & ACTIVE MASTER



medentis medical GmbH

Walporzheimer Str. 48-52 · 53474 Bad Neuenahr/Ahrweiler · Germany

Tel.: +49 (0)2641 9110-0 · www.medentis.de

mm02441_EN

BE SMART. BE ICX.